

# INSTRUCTIONS FOR USE

## Product name

Disposable Safety Syringes (auto-disable)

## Product construction

The syringe includes a barrel, a plunger, a protecting cap, a needle tube, a needle hub, a sealing ring and a piston (ring type).

## Syringe Sizes

0.5ml, 1ml, 3ml, 5ml, 10ml, 20ml

## Needle gauge and length

- Needle gauge: 30G (0.3mm), 29G (0.33mm), 28G (0.36mm), 27G (0.4mm), 26G (0.45mm), 25G (0.5mm), 24G (0.55mm), 23G (0.6mm), 22G (0.7mm), 21G (0.8mm), 20G (0.9mm), 19G (1.1mm), 18G (1.2mm), 16G (1.6mm)
- Needle length: 6mm - 38mm

## Performance and features

- Sterile, pyrogen free
- After injection is finished, pull back the plunger and break the plunger so that the needle will not expose. This will reduce the risk of needle stick injury and prevent re-use of the syringe.







## Intended use and scope of application

This syringe is designed for clinical injections including intramuscular, subcutaneous, intradermal, intravenous injection and blood draws.

## Instructions for use

Clinicians should be familiar with the Instructions for use and be trained in the administration of clinical injections. The proper steps for use are as outlined below:


1. Peel to open the package. Take out the syringe and remove the protecting cap (Fig 1).
2. Insert the needle into the medicine vial to draw medication into the barrel (Fig 2).
3. Expel all air bubbles within the barrel of the syringe and then perform the injection (Fig 3).
4. Depress the plunger all the way within the barrel and when the fiducial line covers with the zero graduation line, the needle hub (together with the needle tube) will attach on the front of the plunger, pull back the plunger (Fig 4.A) so that the needle tube is completely covered in the barrel, then break the plunger (Fig 4.B).
5. Please discard the used syringe in the container for medical consumable waste.

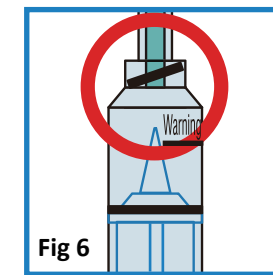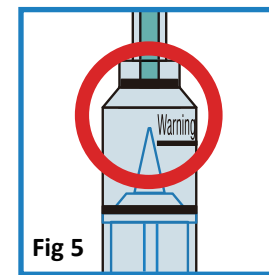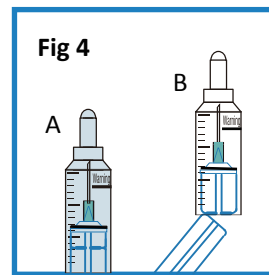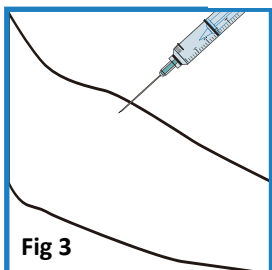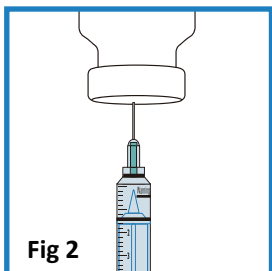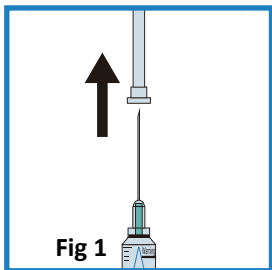

## Caution

1. Please read the "Instructions for use" section before use.
2. Do not use if package is opened or damaged.
3. The syringes are sterile until removed from their protective packaging. Single use only.
4. Do not push the plunger over the "warning" line before injection (Fig 5).
5. Inject slowly. Overexert when doing injection may unseat the sealing ring to cause leakage (Fig 6).
6. Please operate according to the clinical standard of drawing medication from the medicine vial. Please make sure to insert the needle into the "penetration area" of the vial stopper (Fig 7). If the needle is inserted into the area beyond the "penetration area", the needle hub may be moved and the syringe may not be used as expected.

## Symbols

1. Single use only. 
2. Sterilized using ethylene oxide. **STERILE EO**

## Storage

To be protected from moisture and heat. Should be kept indoors, without corrosive gas, dry, well ventilated and clean.

 **Expiry** 3 years

**CE** 0197

## Manufacturer

Shantou Wealy Medical Instrument Co., Ltd.  
North Jinhuan Road (near Qishan Mid-School), Shantou, China  
Postcode: 515064  
Telephone: +86 754 82121747, 82121967, 82121767  
Fax: +86 754 82121654

**EC REP**

## European Authorized Representative:

Lotus NL B.V.  
Address: Koningin Julianaplein 10,  
le Verd, 2595AA, The Hague,  
Netherlands.  
Tel: +31645171879 (English)  
+31626669008 (Dutch)

